

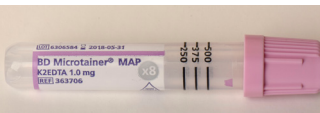
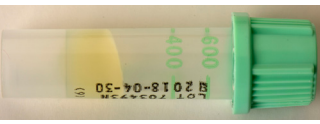
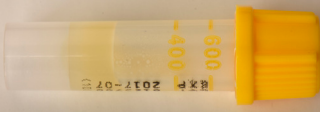
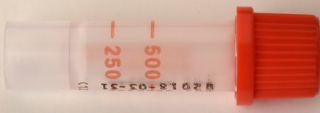






Tube colour	Tube name	Common usage	Instructions
	<b>35uL CAPILLARY TUBE</b>	Capillary glucose	Immediately after collection insert a mixing flea, cap the ends and drag a magnet 10 x the entire length of the tube to mix the sample.
	<b>100uL CAPILLARY TUBE</b>	Capillary blood gas	Immediately after collection insert a mixing flea, cap the ends and drag a magnet 10 x the entire length of the tube to mix the sample.
 CBC - 300mL	<b>EDTA</b>	CBC, Reticulocytes, HbA1c, ESR, Direct Coombs, Haemoglobinopathy studies	Mix tube by gentle inversion 8 - 10 times.
	<b>LITHIUM HEPARIN AND GEL FOR PLASMA SEPARATION</b>	Routine biochemistry. FT4, TSH, Ft3, Ferritin, Cortisol, Progesterone, LH, FSH, Testosterone,, B12, folate, SHBG, Insulin, C-Peptide, AFP, CEA, HCG, Troponin T, Lipids	Mix tube by gentle inversion 8 - 10 times.
	<b>CLOT ACTIVATOR AND GEL FOR SERUM SEPARATION</b>	Virology, Immunology, Lithium, Protein electrophoresis, Ca125, Ca19-9, Ca 15-3, PTH	Mix tube by gentle inversion 5 times.
	<b>NO ADDITIVE</b>	Routine drugs, Infectious mononucleosis	Mix tube by gentle inversion 5 times.

Lancet	Name	Baby size / Patient	Tests
	<b>PINK QUICK HEEL LANCET</b>	<2 Kg	All micro collects.
		>2Kg	Capillary glucose.
	<b>GREEN QUICK HEEL LANCET</b>	>2 Kg	All micro collects on premature babies.
	<b>PINK BDH LANCET</b>	<5Kg	Micro collects on children. >1 year old or walking (whichever comes first)
	<b>BLUE BDH LANCETS</b>	>5Kg	Micro collects on adults.