





URINE COLLECTION PRODUCTS FOR ALL WAIKATO HOSPITAL

Label at bedside: Laboratory request form and samples must match the patient

| Container | Container Name/Code | Urine Test | Minimum volume / Sample Type |
|---|---|---|--|
|  | Tube URINE 16x1000UA BEIGE Oracle Code:196836 DEVICE URINE TRANSFER STRAW Oracle code:196837 | MICROBIOLOGY MC/S (microscopy and culture) | 5mL Midstream Urine or Clean catch - Catheter |
|  | Yellow Top Supplier item: LBS30004Y Oracle Code: 181940 | BIOCHEMISTRY e.g. Protein/creatinine ratio, osmolality, protein, glucose, amino acids and organic acids, micro-albumin, (refer to laboratory handbook for all other test collection requirements) Porphyrin - Protect form light pH – In a syringe | 10 mL Random urine (Not 24 hrs) |
| | | TOXICOLOGY | 10 mL Random urine |
| | | BENCE JONES PROTEIN (BJP) / Electrophoresis | >1.0mL Random urine |
| | | CYTOLOGY | Not first catch 25-50mLs |
|  | 500mL with cap Oracle code:122413 | HAEMATOLOGY Haemosiderin | 10mL Random urine |
| | | TB CULTURE - URINE | Early morning – Full void urine |
|  | Multi-collect tube Orange Top Oracle code:154519 | CHLAMYDIA / GONORRHOEA / TRICHOMONAS PCR | First catch urine To window on side of tube |

Use as a guide only: For all lab tests refer to the **laboratory tests user guide**.