

Winter 2018

RESPIRATORY VIRUS SWABS

For adults and children, one nasopharyngeal swab.


NASOPHARYNGEAL SWABS

Oracle code 212550

Rest back of patient's head against wall or bed.
Insert swab 4-5 cm into nose in horizontal direction.

Rotate swab 3 x each way.

Pertussis testing can be performed on this swab

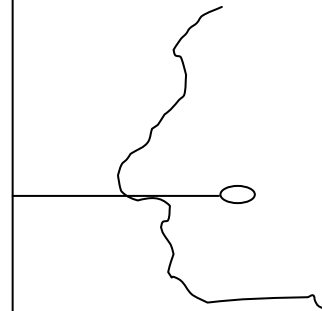

Nasopharyngeal
Viral Swab 2018




Fine
section
for
nose

Snaps
off here



Skin Viral
(thick shaft)

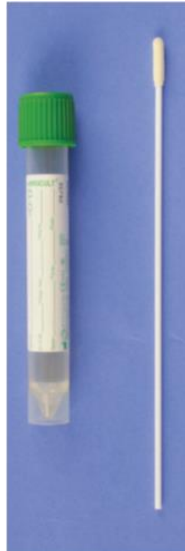

Old Blue
Flexi Wire



Bacterial



“Throat” swabs will **NOT** be tested for virus.

“Nasal” and “Low Nose” swabs will **NOT** be tested.

Chris Mansell 021 833 783
Carl Gale Ext 98530
Jenni Lindeman Ext 98637