May 2022 STANDARD 4 TARGET RESPIRATORY VIRUS SWABS For adults and children, one nasopharyngeal swab.

RSV Influenza A & B SARS-CoV-2

NASOPHARYNGEAL SWABS

Rest back of patient's head against wall or bed. Insert swab 4-5 cm into nose in horizontal direction.

Rotate swab 3 x each way.

COVID and Pertussis testing can be performed on this swab – Only if requested at the same time. Sorry we are unable to add on requests later.





Nasopharyngeal Viral Swabs 2021 Purple or other colour top

"Virus Transport Medium"



Fine section for nose

Snaps off here



Throat Viral (thick shaft)







Bacterial



"Throat" swabs will **NOT** be tested for virus. Please label the swab as "Nasopharyngeal" or "NPS".

Chris Mansell 021 833 783 Jenni Lindeman Ext 98637