

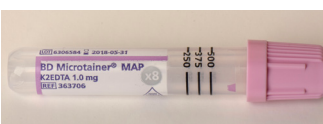
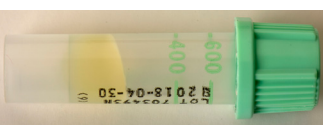
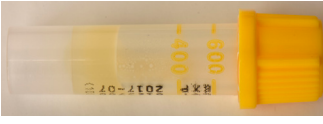
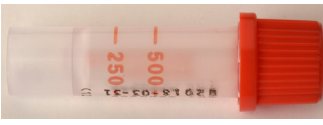






Tube colour	Tube name	Common usage	Instructions
	35uL CAPILLARY TUBE	Capillary glucose	Immediately after collection insert a mixing flea, cap the ends and drag a magnet 10 x the entire length of the tube to mix the sample.
	100uL CAPILLARY TUBE	Capillary blood gas	Immediately after collection insert a mixing flea, cap the ends and drag a magnet 10 x the entire length of the tube to mix the sample.
 CBC - 300µL	EDTA	CBC, Reticulocytes, HbA1c, Direct Coombs, Haemoglobinopathy studies	Mix tube by gentle inversion 8 - 10 times.
	LITHIUM HEPARIN AND GEL FOR PLASMA SEPARATION	Routine biochemistry. FT4, TSH, FT3, Ferritin, Cortisol, Progesterone, LH, FSH, Testosterone,, B12, folate, SHBG, Insulin, C-Peptide, AFP, CEA, HCG, Troponin T, Lipids	Mix tube by gentle inversion 8 - 10 times.
	CLOT ACTIVATOR AND GEL FOR SERUM SEPARATION	Virology, Immunology, Lithium, Protein electrophoresis, Ca125, Ca19-9, Ca 15-3, PTH	Mix tube by gentle inversion 5 times.
	NO ADDITIVE	Routine drugs, Infectious mononucleosis	Mix tube by gentle inversion 5 times.

Lancet	Name	Baby size / Patient	Tests
	PINK QUICK HEEL LANCET	<2 Kg	All micro collects.
		>2Kg	Capillary glucose.
	GREEN QUICK HEEL LANCET	>2 Kg	All micro collects on premature babies.
	PINK BDH LANCET	<5Kg	Micro collects on children. >1 year old or walking (whichever comes first).
	BLUE BDH LANCETS	>5Kg	Micro collects on adults.